

Micro·ZymeTM

Dual Enzymatic Detergent



*Micro-Zyme*TM

Dual Enzyme Detergent Concentrate

**Specially formulated for cleaning of instruments
and medical/ surgical/ dental devices**

DESCRIPTION: Designed for the removal of proteinaceous matter from instruments and medical devices. For use as pre-soak, manual cleaning, ultrasonic cleaning and washer-sterilizer equipment. *Micro-Zyme* dual enzyme quickly and effectively dissolves blood, tissue, mucous and other soils.

SAFETY: Special conditioning agents make *Micro-Zyme* effective in hard or soft water and minimize scale accumulation on instruments and utensils. This product is safe for use on fiber-optics, flexible endoscopes and other medical instruments. Will not stain instruments made of glass, plastic, metal or rubber. Can be safely disposed of down the sink. As with all enzyme products, do not splash into eyes or directly onto skin.

EASY TO USE: Add one to two ounces of *Micro-Zyme* concentrate for each gallon of warm to hot tap water. Amount of product required depends on soil level. Soak and clean as required. Follow by rinsing.

Enzyme Instrument Pre-Cleaner and Transport Gel

**Specially formulated for the pre-cleaning and holding of instruments prior to
cleaning and during transit**

DESCRIPTION: Viscous enzyme gel sprayed directly onto soiled instruments. Coats instrument surfaces, works its way down into cracks and crevices and lock boxes to effectively dissolve blood, tissue, mucous and other soils. Prevents blood from drying on instrument surfaces and sustains moisture. Contains neutral pH enzyme with surfactants and corrosion inhibitors.

SAFETY: Reduces heavy instrument transport weight as compared to liquid-filled trays. Minimizes potential splashing of contaminate debris.

EASY TO USE: Simply spray / apply *Micro-Zyme* Gel directly onto soiled instruments. Cover and allow to stand or transport to destination such as central supply. Rinse and terminally clean using *Micro-Zyme* Detergent. (It is not necessary to rinse before automated cleaning).

PRODUCTS:

Micro-Zyme Dual Enzyme Detergent

4 gallons / case

Order No. MZ04-128

12 quarts / case

Order No. MZ12-032

Micro-Zyme Gel Spray

12 bottles/case

Order No. MZTG12-022

Distributed by:

*Micro-Zyme*TM IS A PRODUCT OF:



For information, Call toll free 1-888-253-2536
Outside USA call 1-847-454-0835
Fax: 1-847-454-0837 email: msi@opticide.com