

# Tono-System™

*...the complete system for cleaning, disinfection  
and storage of tonometer tips and gonial lenses...*



# Tono-System™

**DESCRIPTION:** The *Tono-System* provides the professional eye care specialist with the means of reprocessing contact tonometer tips between eye examinations while following the most current recommendations.

**The convenient TONO-SYSTEM STARTER KIT consists of:**

- **1-Tono-Tray**
- **1-Bottle of Opti-Cide Soak solution**
- **1-Box of Opti-Kleen Wipes**
- **1-Box of Opti-Rinse Wipes**
- **1-Sleeve / Gauze sponge wipes**
- **1-Directions for use sheet and Instructional video tape**



**(Complete Kit Order No. OSK-001)**

#### • DESCRIPTION OF INDIVIDUAL KIT COMPONENTS •

• **TONO-TRAY** consists of a base with a hinged lid to protect against airborne contamination (i.e. dust, etc.) and to contain evaporation of the disinfectant solution. A separate compartment provides for the storage of disposable cleaning, rinsing, and drying wipes. Additionally, the tray contains a separate, removable compartmentalized inner tray for individual soaking of up to five tonometer tips, and a separate compartment for disinfecting & storing gonial lenses. Lastly, the tray contains five sequentially numbered "process" indicators with color-coded markers to indicate which tonometer tips are "ready for use" and which tips are in the "process of disinfection".

(Order no. TT6-001 G)

• **OPTI-CIDE SOAK** is a liquid disinfectant, that when used according to product label directions, meets current criteria for antimicrobial efficacy and is formulated for use on contact tonometer tips and other similar devices. Packaged: Individual bottle / 12 bottles per case. (Order no. OC12-024).

• **OPTI-KLEEN WIPES** are individually packaged wipes saturated with a solution to wipe clean the surfaces of tonometer tips prior to disinfection.

Packaged: 100 per box / 10 boxes per case. (Order no. OKW 10-100B)

• **OPTI-RINSE WIPES** are individually packaged wipes saturated with purified sterile USP grade water used to rinse wipe the surfaces of tonometer tips after disinfection.

Packaged: 100 box / 10 boxes per case. (Order no. ORW 10-100W)

• **GAUZE SPONGE WIPES** made of clean, non-sterile, lint free 2"x 2" gauze are used to dry the surface of tonometer tips after wipe rinsing with Opti-Rinse Wipes.

Packaged: 200 pkg. / 25 pkg. per case. (Order no. GS25-200)

• **INSTRUCTIONAL VIDEO TAPE:** A brief videotape that describes the Tono-System, each of its kit components, and offers a step-by-step in-service presentation on its proper use.

(Order no. TSV-001)

DISTRIBUTED BY:

A PRODUCT OF:



For information call TOLL FREE: (888)253-2536  
Outside USA call: (847)454-0835